

1. Put the condom on after the penis is fully erect and before any contact occurs between the penis and the partner's body.



2. Place the condom on the head of the penis and unroll or pull it all the way to the base.



3. Leave an empty space at the end of condom to collect semen.



4. Remove any air remaining in the tip of condom by gently pressing the air out towards the base of penis.



5. After ejaculation and while the penis is still erect, hold onto the rim of the condom (the open end) so that the condom does not slip off as the penis is carefully withdrawn.



**X TIGHT GERMAN DELAY GEL**  
Male premature ejaculation is the inability to control, or delay, ejaculation. X-tight® Malaysian-made latex condoms have been lubricated with male version of X-tight® German-made herbal delay lubricant jellies. This natural sex alternative provides a mild anaesthetic effect which temporarily desensitizes the penis head. It helps to postpone your climax and extend the lovemaking sessions, leaving you more time to satisfy your partner. Male version of X-tight® German-made natural delay gel is made of herbal analgesic ingredients (such as aloe vera, peppermint) instead of chemical or animal-source ingredients (e.g. benzocaine drug). Therefore, it is recommended for prolong use. Not only it has not any side effects on penis nerves but also it may help to strengthen and firm penis tissue. In addition, male version of X-tight® German-made natural delay gel is available in 50ml tubes and 5ml sachets in the market.



Sexual Health is Wealth!

ENGLISH INSTRUCTION MANUAL  
X-TIGHT NATURAL TOPICAL SEX ALTERNATIVE  
(WOMEN VERSION)



COMBO PACK 3000<sup>+</sup>  
NATURAL LATEX CONDOMS  
WITH SWISS-MADE  
NATURAL GEL




Golden Double Dove  
Made in Switzerland

1. Wash your hand carefully with soap and water . It is highly recommended that women use X-tight® **Swiss-made natural soft soap** which is fully mild and compatible with females' sensitive skin. It is noteworthy that X-tight® natural soft soap takes advantage of "nanotechnology" modern technology. That is more better than ordinary ones available on the open market . Conducted studies prove the product safety and antibacterial effectiveness.

2. For proper sanitary code , we suggest using X-tight® **sterile latex gloves** with "Aloe Vera" coat on its inside layer.

3. Wash the genital area with lukewarm water or , better still, use X-tight® **Swiss-made sterile natural intimate wet wipes** for cleaning your genital area and also give it a pleasant and sweet smell. The unique combination of all natural Italy-made tissues with natural Swiss-made intimate lotion made it perfect.

4. Tear open the sachet of X-tight® Swiss-made Natural Vaginal Gel **at the tear-nick (tear-notch) indicated**. (each consumption is individually sealed in a sachet).

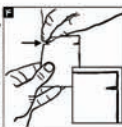

5. Remove a blob of gel from the sachet and put it on your index finger, as shown in the diagram (a **blob of gel is half a sachet**).



6. Stand one of your feet on a raised platform as shown in the diagram and then enter your forefinger into the vagina and gently massage X-tight® vaginal gel into the **first one-third part of it continuously for few seconds**.



7. Massage the clitoral zone with other blob of X-tight® natural gel (the other half of sachet) during foreplay or coition continuously .

☑ Being blessed with the emerging Richter Rubber Technology (founded in 1928) more fully, the product is totally safe and received all the required official approvals. **A century unrivaled professional experience** in sexual medical devices manufacturing, is an absolute quality guarantee of the series of X-tight® natural sexual medical products.

☑ **X-tight® Classification:** USFDA has classified natural vaginal lubricant jellies without condom as a Class I (with latex condom as a Class II) medical device in directive 93/42/EEC.



X-tight®, manufactured by Richter Rubber Technology  
Distributed by Golden Double Dove Inv. Corporation  
P.O. Box: 25111, Los Angeles, California, 90025, USA  
E-mail: FOCS@GDDCORP.COM  
Website: WWW.XTIGHT.CH

Ultra Pleasure @ Extra Lubricant

Free Online Consultation Service